

Silent Night  
Full Face Fit Guide

Brand	Style	Mask Size	Liner Size	Fit Rating	Notes
ResMed	AirFit F10	XS	Small (F101)	Excellent	
ResMed	AirFit F10	S	Small (F101)	Excellent	
ResMed	AirFit F10	M	Medium (102)	Great	
ResMed	AirFit F10	L	Large (F103)	Great	
ResMed	AirFit F20	S	Small (F101)	Excellent	
ResMed	AirFit F20	M	Medium (102)	Excellent	
ResMed	AirFit F20	L	Large (F103)	Excellent	
ResMed	AirTouch F20	S	Small (F101)	Excellent	<a href="#">Align the top edge of the liner with the top of the memory foam.</a>
ResMed	AirTouch F20	M	Medium (102)	Excellent	<a href="#">Align the top edge of the liner with the top of the memory foam.</a>
ResMed	AirTouch F20	L	Large (F103)	Excellent	<a href="#">Align the top edge of the liner with the top of the memory foam.</a>
ResMed	Mirage Quattro	XS	Small (F101)	Excellent	
ResMed	Mirage Quattro	S	Small (F101)	Excellent	
ResMed	Mirage Quattro	M	Large (F103)	Excellent	
ResMed	Mirage Quattro	L	Large (F103)	Excellent	
ResMed	Quattro FX	S	Small (F101)	Excellent	Attach upper Quick Stick tabs under head gear wings.
ResMed	Quattro FX	M	Medium (102)	Excellent	Attach upper Quick Stick tabs under head gear wings.

Silent Night  
Full Face Fit Guide

ResMed	Quattro FX	L	Large (F103)	Excellent	Attach upper Quick Stick tabs under head gear wings.
ResMed	Quattro Air	XS	Small (F101)	Excellent	
ResMed	Quattro Air	S	Small (F101)	Excellent	
ResMed	Quattro Air	M	Medium (102)	Excellent	
ResMed	Quattro Air	L	Large (F103)	Excellent	
ResMed	Ultra Mirage	S	Small (F101)	Excellent	
ResMed	Ultra Mirage	M	Medium (102)	Excellent	
ResMed	Ultra Mirage	L	Large (F103)	Excellent	
Respironics	Amara Full	P	Small (F101)	Excellent	
Respironics	Amara Full	S	Small (F101)	Excellent	
Respironics	Amara Full	M	Medium (102)	Excellent	
Respironics	Amara Full	L	Large (F103)	Excellent	
Respironics	Comfort Gel Full	S	Small (F101)	Excellent	
Respironics	Comfort Gel Full	M	Medium (102)	Excellent	
Respironics	Comfort Gel Full	L	Large (F103)	Excellent	
Respironics	FitLife	S	Small (T101)	Excellent	The FitLife liner was developed specifically for the Respironics FitLife mask.
Respironics	FitLife	L/XL	L/XL (T103)	Excellent	The FitLife liner was developed specifically for the Respironics FitLife mask.
Respironics	Full Life	S	Small (F101)	Poor	Quick Stick tabs do not reach hard plastic.

Silent Night  
Full Face Fit Guide

Respironics	Full Life	M	Medium (102)	Poor	Quick Stick tabs do not reach hard plastic.
Respironics	Full Life	L	Large (F103)	Poor	Quick Stick tabs do not reach hard plastic.
Fisher & Paykel	Forma	S	Large (F103)	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Forma	M	Large (F103)	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Forma	L	Large (F103)	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Flexifit 432	S	Large (F103)	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Flexifit 432	M	Large (F103)	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Flexifit 432	L	Large (F103)	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Simplus	S	Small (F101)	Excellent	
Fisher & Paykel	Simplus	M	Medium (102)	Excellent	
Fisher & Paykel	Simplus	L	Large (F103)	Excellent	
Drive	Wizard 220	M	Large (F103)	Excellent	
Drive	Wizard 220	L	Large (F103)	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic (preferred attaching location) on the bottom side of the mask. Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Hoffrichter	Cirri	S	Small (F101)	Excellent	
Hoffrichter	Cirri	M	Medium (102)	Excellent	
Hoffrichter	Cirri	L	Large (F103)	Excellent	
Hoffrichter	Comfo	S	Small (F101)	Excellent	The Quick Stick tabs need to engage with hard plastic on the mask for best attachment.
Hoffrichter	Comfo	M	Medium (102)	Great	The Quick Stick tabs need to engage with hard plastic on the mask for best attachment.

Silent Night  
Full Face Fit Guide

Hoffrichter	Comfo	L	Large (F103)	Great	The Quick Stick tabs need to engage with hard plastic on the mask for best attachment.
Sleepnet	Ascend	S	Small (F101)	Excellent	Align the top edge of the liner with the top of the gel cushion.
Sleepnet	Ascend	M	Medium (102)	Excellent	Align the top edge of the liner with the top of the gel cushion.
Sleepnet	Ascend	L	Large (F103)	Excellent	Align the top edge of the liner with the top of the gel cushion.
Sleepnet	Mojo	S	Small (F101)	Excellent	Align the top edge of the liner with the top of the gel cushion.
Sleepnet	Mojo	M	Medium (102)	Excellent	Align the top edge of the liner with the top of the gel cushion.
Sleepnet	Mojo	L	Large (F103)	Excellent	Align the top edge of the liner with the top of the gel cushion.

Please note: The Silent Night warranty only applies to masks with fit rating of Good, Great or Excellent.